



Gaumard®
Simulators for Health Care Education



Super OB SUSIE® S500.300

Childbirth Training Torso

- Anatomically accurate pelvic landmarks
- Includes full-term neonate featuring palpable landmarks and articulating limbs
- Lifelike cephalic, breech, shoulder dystocia, operative, and C-section deliveries
- Leopold's, version, PPH management, episiotomy repair, and more

Hands-on obstetric skills training

The new OB SUSIE® is a realistic childbirth torso designed to aid educators in teaching labor and delivery management skills to learners of all levels.

- Pelvic examination
- Urinary catheterization
- Cephalic and breech delivery
- Operative vaginal delivery
- Shoulder dystocia management
- Umbilical cord management
- Placenta delivery
- Rectal medications



Realistic anatomy

The anatomically accurate pelvis and term neonate allows learners to develop clinical skills transferable to scenario-based exercises.

- Adult-sized lower torso with hip-joint articulation
- Uniform skin with realistic elasticity and soft feel
- Pelvic landmarks include ischial spines and coccyx
- Full-term neonate with articulated limbs (18.5 in, 6 lb)
- Neonate landmarks include fontanelle and sutures

Customizable and adaptable

OB SUSIE's modular design allows you to add new functionalities to meet specific training needs. Choose from a growing list of optional packages.

- PPH management
- Episiotomy repair
- Delivery Assist Mechanism

Routine delivery Competencies

- Vaginal examination
- Fetal presentation
- Cardinal movements
- Delivery maneuvers
- Cord management
- Placenta delivery



Palpable pelvic landmarks and dilating cervix



Illustrate descent, rotation, and expulsion




Removable abdomen offers an internal view of the birth

Cord and placenta management

- Nuchal cord
- Cord prolapse
- True knots
- Clamp and cut
- Retained placental fragments



Umbilical cord clamping and cutting

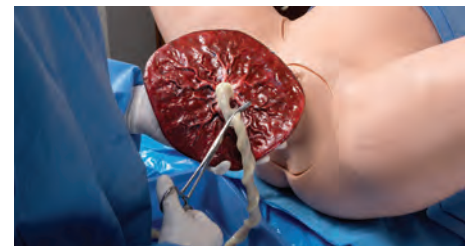

Placenta delivery and examination



Obstetric Emergencies

OB SUSIE can illustrate realistic shoulder dystocia situations to train advanced delivery maneuvers, teamwork, and communication.

- McRobert's maneuver
- Suprapubic pressure
- Rubin
- Woods' screw
- Arm sweeps



Articulated joints provide realistic resistance

Breech and operative delivery

The neonate's smooth skin and articulated limbs support the use of real instruments and advanced maneuvers.



Breech maneuvers: Pinard's, Mauriceau, Ritgen's, Lovset



Forcep indication, application, and traction



Vacuum cup application, suctioning, and traction

PPH management package

- Practice/master postpartum care and emergency management skills with the optional PPH package.
- Fundal and bimanual massage
- Uterine tamponade placement
- Urinary catheterization
- Suppository administration



Soft abdomen and adjustable uterine tone for massage training



Cervix and uterus allow for tamponade placement and hemorrhage control

OB SUSIE® Childbirth Training Torso

S500.300.PK

The OB SUSIE Childbirth Torso includes everything to get started in one easy-to-use package.



Torso Features

- Adult-sized lower torso: diaphragm to upper legs
- Smooth and supple skin
- Removable pregnant abdominal cover
- Modular cervix and birth canal
- Bony landmarks including ischial spines and coccyx
- Lifelike placenta with detachable umbilical cord
- Patent urethra with bladder access
- Integrated urine bladder (400 mL)
- Integrated blood reservoir
- Articulating hip-joints
- Patent rectum

Neonate Features

- Full-term size and weight (18.5 in, 6 lb.)
- Soft full-body skin
- Nasal and oral cavities
- Palpable fontanelle and sutures
- Articulated limb joints
- Detachable umbilical cord

Package Contents

- OB SUSIE Torso
- Pregnant abdomen
- Full-term neonate
- Placenta
- (2) Umbilical Cord
- (2) Umbilical Stumps
- (2) Cervices
- (2) Birth Canals
- Mineral Oil
- Talcum Powder
- Urine fill kit

PPH Management Package

S500.300.117

The PPH Management Package enables the training of postpartum care and emergency management skills.



Features

- Realistic post-delivery perineum
- Uterine tamponade placement
- Adjustable uterine tone
- Urinary bladder catheterization
- Suppository administration

Package Contents

- PPH Perineum
- Boggy Uterus
- PPH Cervix
- Blood concentrate
- Blood fill kit

Delivery Assist Mechanism

S500.300.929

The Delivery Assist Mechanism manual lever makes controlling descent and rotation virtually effortless.



Features

- Controlled descent
- Fetal rotation control
- Hold during descent

Package Contents

- Birthing mechanism
- Rear panel
- Fasteners set

Postpartum Episiotomy Suture Trainer

S500.300.133

The episiotomy repair trainers simulate human tissue and support the use of real sutures and instruments.



Features

- Realistic soft skin
- Supports real sutures
- Durable self-healing skin

Package Contents

- Mediolateral left episiotomy perineum
- Midline episiotomy perineum

Request a Quote

www.gaumard.com/quote

sales@gaumard.com

Toll-Free USA & Canada

1.800.882.6655

Worldwide 305.971.3790